

SynAtomy™ Platform Pump Quick Start Guide


A-PPM-A-0005

Package Contents

Tray x1
Tray top x1
Battery Pack x1
Submersible Pump x1
Supply tube x1
Return tube x1

Tray and Pump Set-up


Step 1: Insert each blue tube through the larger holes on each side of the tray. Connect the tube to the pump as shown in the picture below. Place the pump's wire through the cut-out section like shown in the picture.

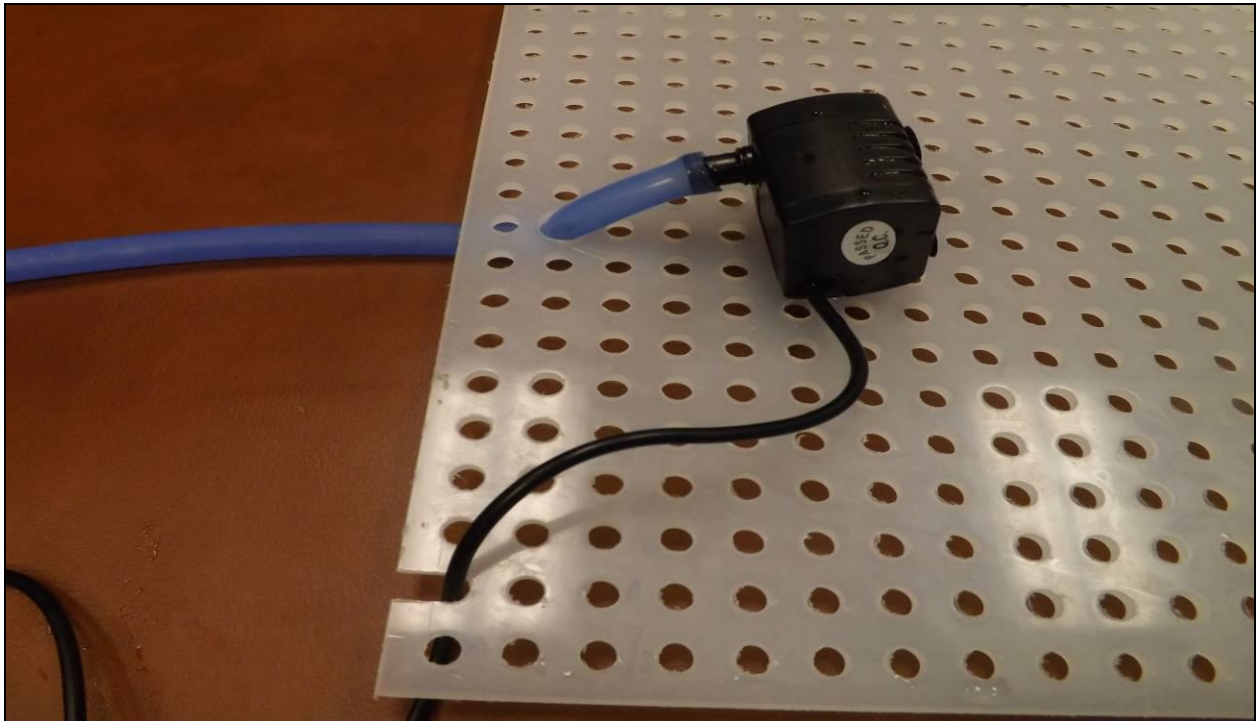

Step 2: Fill the tray with water, and then place the tray top on the tray. Submerge the pump in the water. Have its wire come up outside the cut-out by the corner.




Step 3: Place the pad on the tray and align the vessel Luer fittings with the tube Luer fittings.

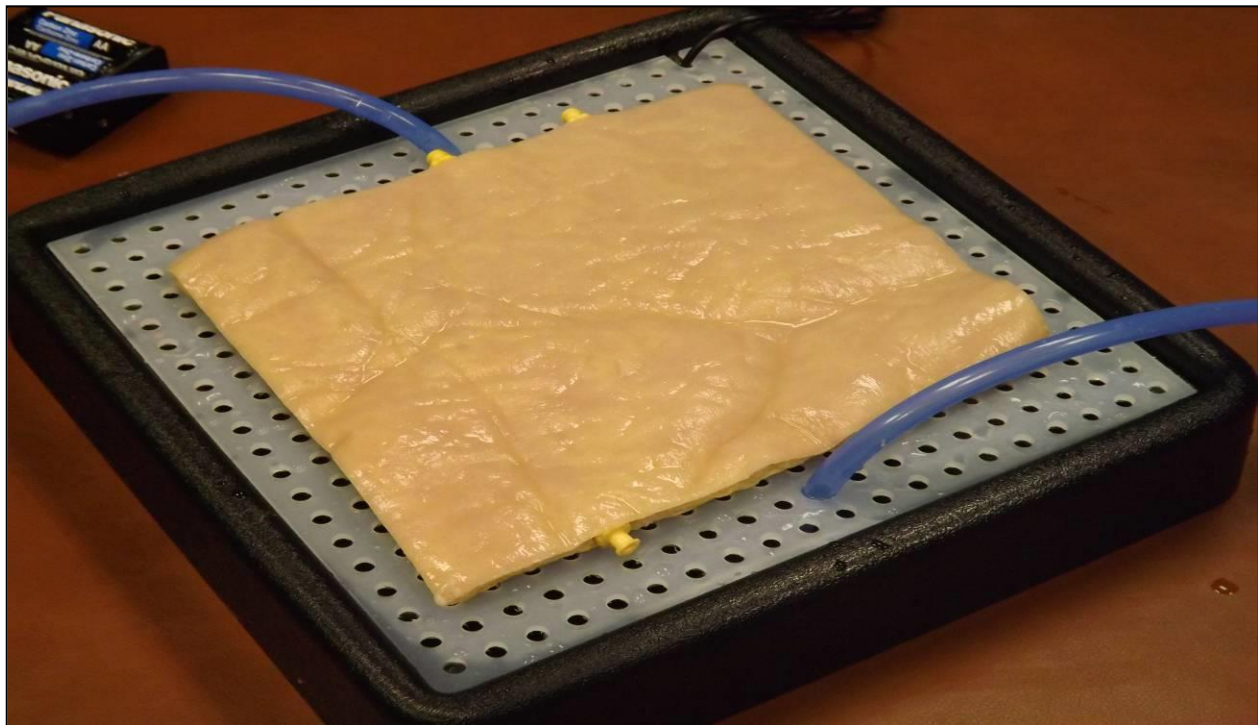



Step 4: Twist and lock the Luer fittings to attach both tubes to the vessel.

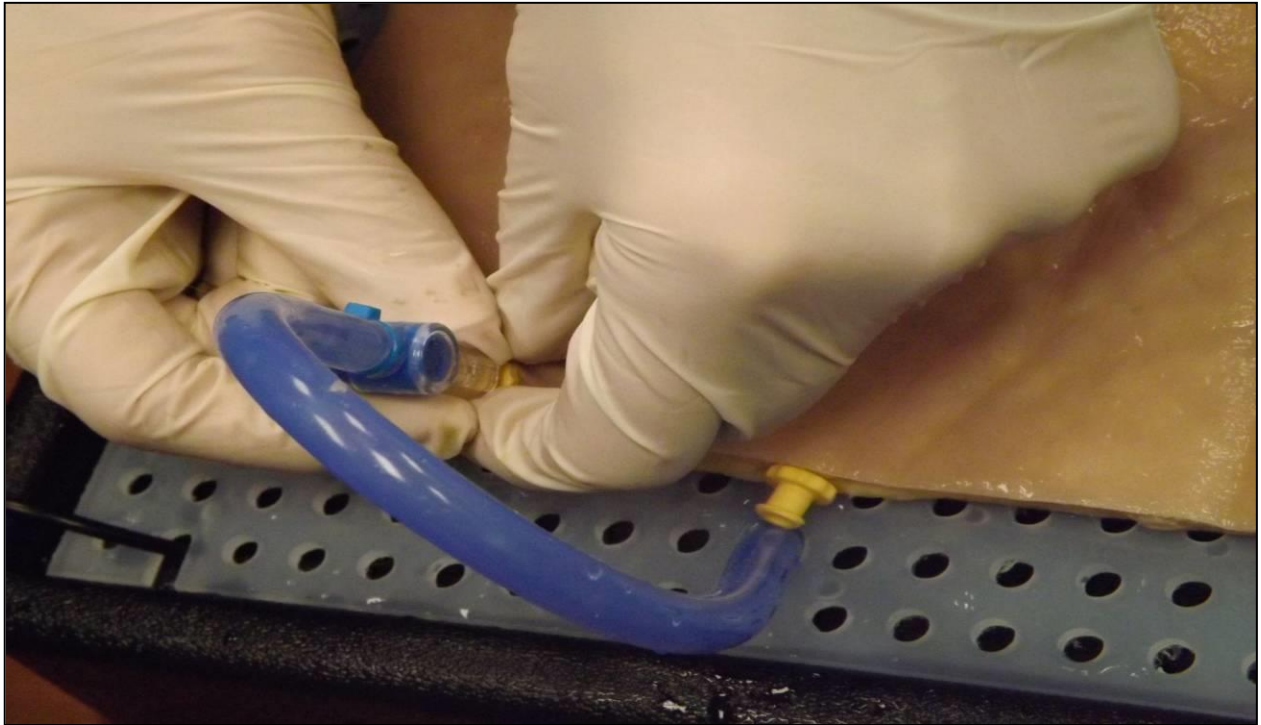

Step 5: Plug together the battery pack with the pump cable to start the pump.



Power Supply and Plumbing

Plug in the power chord to an outlet and to the pump. Flip the power switch located on the pump to start plumbing.

Handling and Storage

All SynDaver™ Labs soft tissue products are made from water, fibers, and salts. Our oldest retained products are nearly twenty years old and this model will last several decades as long as it remains hydrated. If the optional storage case was not purchased with the model the soft tissues should be stored in a plastic container or large bag with a small amount of excess water.

Best practice also involves wearing gloves during use to keep the model clean. In any case, always wash the model with soap and water prior to storage.