

SynAtomy Arthrocentesis Trainer

160640

Quick Start Guide

Package Contents

Arthrocentesis Trainer
60mL Syringe
Foam Stand



Step 1:

Place the skin on the knee. If you can feel any air pockets, massage them out.

(Arthrocentesis Trainer in use)



Step 2:

Attach the lower port on the trainer to your luer lock syringe.



Step 3: For aspiration of fluid, depress the syringe.

See the SynDaver Care and Storage Guide for care and storage of your SynDaver trainer. A print out of the guide is included in every delivery and available on our website at:

syndaver.com/care