

Using Consumable Item(s)

Thank you for your purchase. Your consumable item(s) may remain in original packaging until ready for use or up to one year. When ready for use, follow the instructions below:

1. Over a sink, cut open the plastic package and pour the shipping fluid down the drain.
2. Wear gloves to protect the tissue from microorganisms on your hands.
3. Take the product out of the packaging and bathe it with any hand soap and water.
4. While in use, place the item on clean surfaces and handle with gloves.
5. Used soft tissue is safe to dispose of with other non-recyclable material.

Reusing your Consumable Item(s)

While consumable items are meant to be used in one session, they may also be stored and reused repeatedly if desired. For storage, follow the instructions below:

1. Remove any metal such as surgical staples, which may rust. Sutures and non-metal implants are safe to leave within the tissue.
2. Bathe the tissue gently with any hand soap and water. Rinse thoroughly.
3. Place the tissue inside a sealable plastic container or zip-top bag.
4. Add a drop of hand soap inside and fill with enough water to cover the tissue. If storing in a zip-top bag, exclude as much air as possible.
5. Store at room temperature away from direct light or inside a refrigerator.

Storing Multiple Items Long-Term

SynDaver supplies small to extra large storage systems for higher volumes of synthetic soft tissues. Write to us at care@syndaver.com or call us at 813-600-5530 for more information.

For More Information About SynDaver Products

813.600.5530 • [SynDaver.com](https://www.SynDaver.com)

SynDaver
Beyond Human

 **MANUFACTURED IN THE USA** • 8506 Benjamin Road, Tampa, FL 33634